Mondocel[®] Ear Tampon

INSTRUCTIONS FOR USE (EN)

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BRUKSANVISNING (SV)

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Mondocel[®] Ear Tampon

Instructions for use – ENGLISH

Contents

1. INTENDED USE
2. COMPOSITION
3. INDICATIONS FOR USE
4. CONTRAINDICATIONS
5. WARNINGS
6. PRECAUTIONS
7. PRODUCT DISPOSAL
8. PRINCIPLES OF OPERATION

1. INTENDED USE

Mondocel[®] Ear tampons are intended to be temporarily placed into the external acoustic meatus to absorb body fluids and secretions by excellent absorption capacity of the tampon and to reduce edema by their compression capacity. The tampons are compressed so that they can easily be inserted into the auditory meatus. These compressed devices expand rapidly with hydration and adapt well to the anatomical site and assist in tamponade.

Mondocel[®] Ear tampons have a spongy, porous structure and are fiber-free.

2. COMPOSITION

Polyvinyl alcohol (PVA) foam

Mondocel[®] Ear tampons are available in different sizes, with or without string or central lumen.

3. INDICATIONS FOR USE

- Postoperative after ear surgery (tympanoplasty)
- Treatment of inflammation of the external acoustic meatus (Otitis externa)

Mondocel[®] Ear tampons should only be used by medical staff.

4. CONTRAINDICATIONS

No known contraindications.

5. WARNINGS

1) The use of internal tamponade devices can in rare cases and in particular conditions, perhaps following surgery, present a risk of **toxic shock syndrome (TSS)**. Toxic shock syndrome is a multisystem inflammatory

response due to infection by bacteria capable of producing certain endotoxins. Mondocel® Ear tampons do not contain any antibiotics and exhibit no antimicrobial properties. They are not bacteriostatic towards pre-existing infection, nor do they prevent the occurrence of new infections. In the case of pre-existing infections it is recommended that appropriate treatment be instituted.

2) Mondocel® Ear tampons must be kept moist during entire duration of use.

6. PRECAUTIONS

1) Do not use the tampons if package is breached or damaged.

2) Mondocel® Ear tampons are supplied sterile, do not re-sterilize.

3) Do not apply alcoholic solutions on the tampon as this compromises the integrity, composition and functionality of the foam.

4) Mondocel® Ear tampons are for single use only; reuse of the device can lead to several hazards:

- cross contamination
- infection risks
- deterioration and alteration of the characteristics of the device due to the first use with consequent risk of injury to the patient

5) Do not use the tampons after the expiry date.

6) Keep the tampons away from sunlight and heat. Keep the tampons dry during storage and store them between 5°C and 35°C.

7. PRODUCT DISPOSAL

In order to minimize the risk of potential infection hazards or environmental pollution Mondocel® Ear tampons should follow disposal procedures according to applicable and local laws, rules, regulations and infection prevention standards.

8. PRINCIPLES OF OPERATION

Use clean or aseptic techniques for application, according to your institutional protocol.

Insertion

The tampon is inserted in dry, compressed state manually or with the help of a pair of tweezers into the auditory meatus. Lubricate the distal edge with water or saline based lubricant, leaving proximal end uncoated for hydration.

In order to prevent ingrowth after tympanoplasty, it is urgently indicated to use a wound contact layer prior to inserting the ear tampon or increase the frequency of change.

Some ear tampons contain a string. This string, in addition to being safety factor, facilitates the process of removing the tampon. The string needs to be secured to the cheek by means of a plaster.

Expansion

Once inserted, the ear tampon is hydrated with saline or aqueous solution by means of a syringe.

The temperature of the hydration solution can affect the expansion rate. A cooler hydration solution results in a slower expansion rate, a warmer hydration solution results in a faster expansion rate.

The tampon must kept moist while in use. Depending on the individual situation, the tampon must be hydrated one or more times a day, in order to prevent drying and consequent loss of elasticity of the ear tampon.

Removal

Mondocel® Ear tampons can be left in situ for one to three days in case of Otitis externa and in any other case up to a maximum of five days.

In case of tympanoplasty the ear tampon may be left in the ear for up to three weeks. When the ear tampon remains in place for three weeks (after tympanoplasty), the treating physician should decide whether to administer suitable medication in order to prevent infections.

Before removal, the tampon must be hydrated with either sterile water or saline (wait five to ten minutes for proper hydration).

Use a pair of tweezers to grasp the tampon and gently remove it.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer, who will take care for reporting the applicable Competent Authority.

 Mondomed NV
Middenweg 20
BE-3930 Hamont-Achel

CE 1639

Phone: +32 11 440 990
Email: info@mondomed.be
<http://www.mondomed.be>

	European conformity		Do not resterilize
	Medical device		Do not use if package is damaged
	Manufacturer		Consult instructions for use
	Date of manufacture		Keep away from sunlight
	Use-by date		Keep dry
	Batch code		Temperature limit
	Catalogue number		Unique device identification
	Sterilized using irradiation		Green dot
	Double sterile barrier system		Dispose of packaging in an environmentally responsible way
	Do not re-use		

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